ADSEAL Plus

Root Canal Sealer

Manufacturer

META BIOMED CO., LTD.

270, Osongsaeangmyeong 1-ro,
Osong-eup, Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do, Republic of Korea

Tel +82-43-216-0433

Fax +82-43-217-1988

E-Mail info@meta-biomed.com

www.meta-biomed.com

META BIOMED

SYMBOLS

Manufacturer

nl Fabrikant
fr Fabricant
de Hersteller
it Produttore
pt Fabricante
es Fabricante
tr Üretici

Date of manufacture

nl Productiedatum
fr Date de fabrication
de Herstellungsdatum
it Data di produzione
pt Data de fabrico
es Fecha de fabricación
tr Üretim tarihi

Batch Code

nl Partijcode
fr Code de lot
de Chargennummer
it Codice Lotto
pt Código de lote
es Código de lote
tr Parti Kodu

CE - Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC

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EC REP

Authorized representative in the EC

nl Erkende vertegenwoordiger in de EG
fr Représentant autorisé dans l'UE
de Bevollmächtigte/r in der EU
it Rappresentante autorizzato nella CE
pt Representante autorizado na UE
es Representante autorizado en la CE
tr AB'de yetkili temsilci

INSTRUCTION FOR USE

Root Canal Sealer

ADSEAL Plus IS A RESIN-BASED ROOT CANAL SEALER OF THE HIGHEST QUALITY

Direction for using ADSEAL Plus

ADSEAL Plus is a resin based root canal sealer which is a paste-paste type of dual syringe. It has an outstanding chemical and physical property including extremely excellent sealing property and biocompatibility. ADSEAL Plus complies biocompatibility with the requirements of ISO 6876 for dental root canal sealing materials.

Intended Purpose

ADSEAL Plus is a biocompatible root canal sealer for permanent sealing of root canals following established endodontic procedures and may be used in conjunction with the auxiliary materials in the root canal (i.e. gutta percha points).

Indication

ADSEAL Plus used for the permanent containment of the root canal, regardless of the combined use of endodontic filling points. This material is intended to be used for root canal filling from the crown side.

Main Composition

- Base

Epoxy oligomer resin
Ethylene glycol salicylate
Calcium phosphate
Zirconium oxide

- Catalyst

Poly aminobenzoate
Triethanolamine
Calcium phosphate
Zirconium oxide
Calcium oxide

Characteristics

When using ADSEAL Plus in combination with gutta-percha points, this material shows the following properties:

1. Easy to mix by paste:paste type
2. Hermetic sealing ability
3. Nonstaining to teeth
4. Insoluble in tissue fluids
5. Excellent biocompatibility
6. Good radiopacity
7. Flowability is more than 17mm
8. Setting time is less than 90 minutes at 37°C (99°F)

How to Operate or Use the Product:

1. Preparations before Use

- 1.1 Check the product for any abnormality such as leaks, discoloration, etc.
- 1.2 Familiarize yourself with how to use this product and read the warnings especially its side effects and the prohibitions through the user manual.
- 1.3 Check the expiration date of the product.
- 1.4 Use a rubber dam for mouth protection.

2. How to Use the Product

- 2.1 A root canal is formed using a dental instrument (File).
- 2.2 After removing the remains in the root canal completely, the root canal is sterilized and washed with hypochlorous acid or EDTA solution, and dried at the paper point.
- 2.3 Measure the length of the root canal with a root canal length measuring instrument and a file, then select a suitable gutta-percha master point.
- 2.4 Remove the brown syringe cap from the ADSEAL Plus syringe. Securely attach the green dispense tip to the syringe, push the syringe plunger to see if the material is coming out of the gauze and wipe it off. Squeeze in a 2: 1 ratio on a mixing pad and mix for 20 seconds using a spatula without air bubbles.
- 2.5 Open the brown cap of the ADSEAL Plus Auto-mix and push the syringe plunger to make sure that the same amount of material is coming out of the gauze, wipe it off, install the auto mixing tip and squeeze it into the mixing pad to make sure it is mixed with uniform color without bubbles.
- 2.6 Apply ADSEAL Plus mixed on the mixing pad to the end of the tip of the GP point and

- slowly pushed into the root canal to fill ADSEAL Plus.
- 2.7 After filling is complete, remove excess ADSEAL Plus with a bundle of alcohol syrup or burrs before curing, and cut out the upper part using a dental instrument (Heated plugger).
 - 2.8 Take dental X-rays and check its overall condition.
 3. How to Keep and Manage It after Use
 - 3.1 After each application, place the dispense tip cap tightly onto the syringe immediately. *The green dispense tip should NOT be removed and MUST be kept attached for continual use.
 - 3.2 Auto-mix & Endo tips are to be used only once and should be discarded after use. (Reusing is prohibited.)
 - 3.3 Store in a place not exposed to direct sunlight.
 - 3.4 Do not store near fire.
 - 3.5 Keep out of reach of children.

Points of Caution for Use

1. General Points of Caution

- 1.1 This product is designed to be used only for oral tissues. Please be careful to avoid eye contact. In the event of eye contact, wash the contacted eye immediately and then consult a ophthalmologist.
- 1.2 This product must not be used for internal use.

2. Warnings

- 2.1 It is prohibited to use this product for any purpose other than its intended purpose or for a non-expert to use it.
- 2.2 This product should be kept out of the reach of children.
- 2.3 Do not use this product for any patient with a history of hypersensitivity to it, such as a rash or dermatitis.

3. Points of Caution for Application

- 3.1 Be sure to install a rubber dam in the patient's mouth before using this product.
- 3.2 If the patient shows any hypersensitivity reaction to this product, such as a rash, during its use, stop using it immediately and have the patient consult a dermatologist.
- 3.3 Be sure to take X-rays according to the treatment procedure to confirm the status of sealing in the root canal.
- 3.4 Do not heat when mixing base and catalyst.
- 3.5 When using apical file or lentulo spiral, be careful not to overfill beyond apex.

4. Side Effect

None Known

5. Contraindications

This product should not be used for the obturation of primary teeth.

How to Store the Product

Avoid direct light and store at room temperature (15~25°C)

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