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Lip Retractor

For professional use only

Carefully read the information contained in this document before using the product. Keep this document for later reference until there is no further interaction between this product and the last patient on which it will be used.

Product Description

Arcflex is a product that promotes the separation of the lips, cheeks, and tongue, allowing wide accessibility and visibility of the operative field during dental procedures. Available in two sizes: medium and large.

Product Availability

Check the product commercial availability in your country.

- Pack containing 1 retractor of a single size (medium or large);
- Pack containing 2 retractors: 1 medium-sized retractor and 1 large-sized retractor.

Basic Composition

Polyacetal.

Product Indication

The product is suited for all situations in which greater exposure of the dental arches is desired with maximum separation of the lips, tongue, and cheeks, covering procedures such as:

- In-office whitening;
- Prophylaxis;
- Application of desensitizers;
- Restorations in anterior teeth;

- Bonding brackets;
- Minor previous gingival surgeries;
- Previous prosthetic preparations;
- Intraoral photographs.

Precautions

- This product is intended to be used only in accordance with its instructions for use;
- The product is provided in a non-sterile condition. Therefore, it must be cleaned and sterilized before each use;
- The composition of the product's original packaging is unsuitable for sterilization. Before sterilization, the product must be repackaged in appropriate packaging for the selected sterilization method;
- Successive uses and sterilizations cause fatigue of the arc and limit its useful life. Test the product by compressing the arc before each use and discontinue it if excessive fatigue or risk of breakage is observed;
- Do not place the Arcflex retractor against the walls of the autoclave, as there is a risk of overheating and consequent product damage.

Contraindications

- Do not use the product if the patient is allergic to the substance present in the composition. In case of allergic reactions, discontinue use of the product and, if necessary, seek medical attention.

Side effects

No adverse effects will occur as long as the indications for use are followed.

Directions for use

Carefully read the **Warnings, Precautions, Contraindications** and possible **Adverse Effects** of this document before using the product.

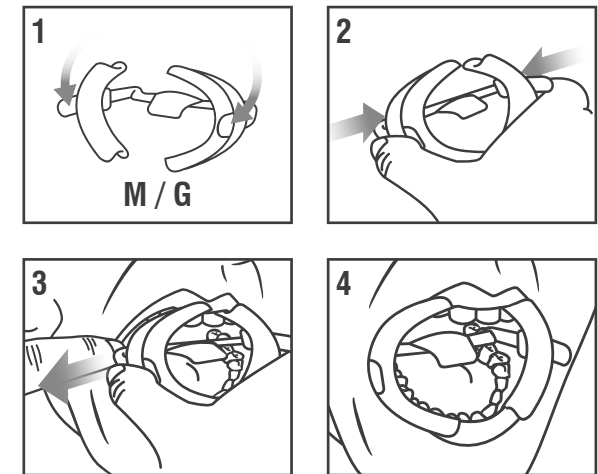
Fig. 1: Select the correct size of Arcflex retractor for the patient.

Note: The product size is printed on the activation vane.

Fig. 2: Hold the retractor by the scoops' activation vanes, pressing them so that one shell meets the other, to streamline the insertion of the product into the patient's mouth. Make sure the tongue controller is facing down.

Fig. 3: Insert the retractor starting from one side of the mouth, keeping the scoops pressed. Fit the upper and lower lips on the side where the insertion began. Insert the main arch on the opposite side and then fit the lips.

Fig. 4: Ask the patient to close the mouth, resting the teeth on the support platforms.

Cleaning and Sterilization

This product is reusable and provided non-sterile.

Therefore, it must be properly sanitized and sterilized before each use.

Cleaning:

1. Immerse the Arcflex retractor for at least 5 minutes in the cleaning solution (Proaction AS 100 4E – 0.5% at 30 °C to 45 °C) so that the product is sufficiently covered in the ultrasonic pan. Make sure there is no contact with other materials.

2. Remove the product from the cleaning solution and wash it intensively at least 3 times for at least 1 minute in running water.

3. Dry the product with a paper towel or soft, absorbent, disposable cloth.

4. Perform visual inspection to verify the effectiveness of the cleaning process and product integrity condition. If dirt is detected, repeat the cleaning procedure.

Sterilization:

Packing Recommendations: Insert cleaned instruments into sterilization trays in single-use sterilization packaging (single or double surgical grade paper packaging) and/or sterilization containers that meet the requirements of EN ISO 11607.

WARNING: Do not autoclave the product in its original packaging.

For sterilization, use only the steam sterilization method, according to the following parameters:

	Fractional vacuum/dynamic air removal ¹ recommended	Gravitational
Sterilization Time	4 minutes	15 minutes
Sterilization Temperature	134 °C/273 °F	134 °C/273 °F
Drying time	At least 20 minutes ²	At least 20 minutes ²

¹At least three vacuum steps.

²The required effectiveness in the drying time depends directly on the parameters that are the user's responsibility (loading and density configuration, sterilization conditions), and these must be determined by the user. However, drying time, where applicable, should not be less than 20 minutes.

NOTE:

- Immediate use/flash sterilization procedure should not be used.

- Do not use dry heat sterilization, radiation sterilization, formaldehyde and ethylene oxide sterilization or plasma sterilization.

Preservation and Storage

- Store the product in a clean, dry environment and in the appropriate packaging for the selected sterilization method;
- Protect the product from direct exposure to sunlight;
- Keep the product at temperatures between 5 °C and 35 °C/41 °F and 95 °F.

Warnings

- This material was manufactured for dental use

only and must be handled in accordance with the instructions for use;

- Do not use the product after its expiration;
- Follow the legislation in your country when disposing of the product;
- Keep away from children.

Expiration date

Printed on the product packaging.

This material has been manufactured for dental use only and must be handled and stored only in accordance with its instructions for use. The manufacturer is not responsible for any damage caused as a result of use not in accordance with the guidelines, improper storage or incorrect handling. Furthermore, the user is obliged to verify and verify, before handling and/or applying the product, and under his/her responsibility, if this material is compatible with the intended use. Furthermore, it should be noted that data descriptions do not constitute any kind of guarantee and, therefore, are not binding.

Legendas de símbolos Leyendas de los símbolos Explanation of symbols	
	Data de fabricação Fecha de fabricación Date of manufacture
	Número do lote Número del lote Batch Number
	Prazo de validade Fecha de validez Shelf life
	Limites de temperatura Límites de temperatura Temperature limits
	Não utilizar se a embalagem estiver danificada No lo utilice si el empaque está dañado Do not use if packaging is damaged
	Conservar seco Consérvelo seco Keep it dry
	Manter afastado da luz solar Manténgalo lejos de la luz solar Keep protected from sunlight
	Fabricante Fabricante Manufacturer
	Consulte as instruções de utilização Consulte las instrucciones de utilización Refer to instructions for use
	Representante na Comunidade Europeia Representante en la Comunidad Europea Representative in the European Community

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