Diamond excel

BR | ES | EN | FR | DE | ZH | IT

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Micronized diamond polishing paste

Professional use only

Read all the information in this manual carefully before using the product. Keep the manual for later reference at least until the product is finished or until there is no more interaction between the product and its last patient.

Product description

Diamond Excel is a polishing paste produced using micronized diamonds with a very fine grit (2 to 4 microns) and extremely hard in order to meet the requirements of polishing and shining porcelain, dental enamel, resins and other restoring materials. It can be used in prosthetic laboratories too. The paste is produced from atoxic ingredients, soluble in water, especially selected to help with the lubrication during the polishing (it minimizes heat generation during the procedure) and facilitates removal at the end of the treatment. The paste has a medium viscosity and is not affected by heat (it does not melt when heated).

Presentation form

Paste containing 1 syringe with 2g of paste.

Basic composition

Micronized diamond, lubricant base,

thickener and emulsifier.

Product indication

Diamond Excel was developed for the obtainment of high shine in all types of dental restorations, especially porcelains, with the help of Diamond or Diamond Flex felt disks.

Precautions and contraindications

Use low rotation and intermittent movements with low pressure applied to the felt on the tooth. This prevents excessive heating of the surface.

Always use new felt to execute a polishing. The use of the same felt disk with abrasives of other pastes can jeopardize the success of the polishing.

Instructions for use

1.Prepare the area to be polished leaving it smooth and in the final format desired. If necessary use sandpaper disks (Diamond Pro Disks granulations G, M, F and/or XF) or medium granulation paste (Diamond ACI and ACII) for the finishing prior to the polishing. This stage is of paramount importance, as obtaining a shine is very dependent on the ideal surface condition.

2.Place a small quantity of paste on a felt disk (Diamond or Diamond Flex Felt Disks) or directly on the tooth or dental restorations with the help of an instrument (spatula) and proceed with the polishing.

3.Repeat procedure 2, if necessary, until obtaining the effect desired.

Conservation and storage

Keep the product in its original package always well sealed. Store the product at 5°C to 30°C/41°F to 86°F. Do not freeze the product.

Warnings

Do not utilize the product after the expiry date. When disposing of the product, follow the legislation of your country. Keep out of children's reach.

This material was manufactured for dental use only and must be manipulated according to the manufacturer's instructions of use. The manufacturer is not responsible for any damage caused by misuse or improper manipulation of the product. The user is obligated to prove (before using the product and under his/her responsibility) that this product is compatible with the desired utilization, especially when such utilization is not indicated in the instructions of use. Data description does not constitute any type of warranty and therefore is not binding.

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