PT | ES | EN

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EN INSTRUCTIONS FOR USE**Diamond polishing paste****For Professional Use Only**

Carefully read all information in this document before using the product. Keep it for reference at least until the product is completely consumed and/or until there is no further interaction between this product and the last patient in which it will be used.

Product Description

Polishing paste produced with micronized diamond of extra fine grit (0.5 micron), soluble in water, used to meet the requirements of polishing and shine of porcelain, dental enamel, resins and other restorative materials.

Commercial Availability

Pack containing 1 syringe with 2 g of paste.

Basic Composition

Micronized diamond, lubricating base, thickener, emulsifier and mint essence.

Product Indication

Indicated for polishing restorative materials in general, especially porcelain, using Diamond or Diamond Flex felt discs.

Precautions

- This product is intended to be used only in accordance with its instructions for use;
- Use low rotation and intermittent movements with low pressure to apply the felt on the tooth. This prevents surface overheating;
- Always use new felts for polishing. Using the same felt disc with abrasives from other pastes can compromise the success of the procedure;
- The use of all Personal Protective Equipment (PPE) is recommended when handling the product.

Contraindications

The product is not indicated for use in patients allergic to its components.

Side effects

No side effects are expected considering proper use of the product.

Directions for use

Carefully read the Warnings, Cautions and Contraindications in this document before using the product.

1) Prepare the surface to be polished leaving it smooth and in the desired final shape. If necessary, use sandpaper disks (Diamond Pro grits G, M, F and/or XF) or medium and fine grit pastes (Diamond ACI and ACII)

for finishing prior to polishing. This step is of fundamental importance, since obtaining gloss is strongly dependent on the ideal condition of the surface.

2) Place a small amount of paste on a felt disc (Diamond or Diamond Flex Discs) or directly on the restorative material with the aid of an instrument (spatula) and polish. After the procedure, rinsing with a water jet and suction is recommended.

3) Repeat procedure 2, if necessary, until the desired effect is achieved.

Preservation and Storage

Keep the product in its original packaging and always tightly closed;

Store the product at temperatures from 5 °C to 30 °C/ 41 °F to 86 °F;

Do not freeze the product.

Expiration date

Printed on product packaging.

Warnings

Do not use the product if expired;

For product disposal, follow your country's legislation;

Keep out of the reach of children.

This material has been manufactured for dental use only and must be handled and stored only in accordance with its instructions for use. The manufacturer is not responsible for any damage caused as a result of use not in accordance with the guidelines, improper storage or incorrect handling. Furthermore, the user is obliged to check and verify, before handling and/or applying the product, and under his/her responsibility, that this material is compatible with the intended use. It should also be noted that data descriptions do not constitute any kind of guarantee, and therefore are not binding.

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EN: Any serious incident occurring with the device must be reported to the manufacturer and the competent authority where the user and/or patient is established.

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