DENTSCARE LTDA - Av. Edgar Nelson Meister, 474 - Distrito Industrial - 89219-501 - Joinville - SC - Brasil / Brazil. CNPJ / Tax ID: 05.106.945/0001-06 INDUSTRIA BRASILEIRA / BRAZILIAN INDUSTRY Responsável Técnico / Technical Authority: Friedrich Georg Mittelstadt CRQ: 13100147 - SC ANVISA nº 80172310003 - Marca / Brand: FGM® / Fabricado no Brasil / Made in Brazil

ec rep NUNO FLORES - Al. Bonifácio Lazaro Lozano, 3 - Piso 0 - C 2780-125 - Oeiras / Portugal Tel: (351) 21 4439292 - ec.rep@fgm.ind.br

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Pasta de polimento

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Polishing paste

For professional use only

Read all the information in this instruction manual carefully before using the product. Store it for consultation at least until the product is fully consumed and/or until there is no further interaction of the product with your last patient.

Product description

Polishing paste based on aluminum oxide with extra fine grit (6 to 8 microns), soluble in water, used to meet the requirements of polishing and shine of resins in general.

Form of presentation

1 syringe with 4 g of the product.

Basic composition

Aluminum oxide, polyethylene glycol, thickener, mint essence and deionized water.

Indication for use

Final polishing of composites in general.

Precautions

- This product is intended to be used only in accordance with its instructions for use.
- Use low rotation and intermittent movements with low pressure to apply the felt on the tooth. This prevents overheating of the surface.
- Always use new felts for polishing. The use of the same felt disc with abrasives from other pastes can compromise the success of the procedure.
- It is recommended to use all personal protective equipment (PPE) when handling the product.

Contraindications

The product is not indicated for use in patients allergic to its components.

Side effects

No side effects are expected considering proper use of the product.

Directions for use

Carefully read the Warnings, Precautions and Contraindications in this document before using the product.

- 1) Prepare the surface to be polished leaving it smooth and in the desired final shape. If necessary, use sandpaper disks (Diamond Pro grits G, M, F and/or XF) or medium and fine grit pastes (Diamond ACI and ACII) for finishing prior to polishing. This step is of fundamental importance, since obtaining gloss is strongly dependent on the ideal condition of the surface.

- 2) Place a small amount of paste on a felt disc (Diamond or Diamond Flex Discs) or directly on the restorative material and polish.

After the procedure, rinsing with a water jet and suction is recommended.

- 3) Repeat procedure 2, if necessary, until the desired effect is achieved.

Storage conditions

Keep the product in its original packaging, always closed.

Store the product at temperatures from 5 °C to 30 °C / 41 °F to 86 °F.

Do not freeze the product.

Expiration date

Printed on the product packaging.

Warnings

Do not use the product if it is out of date.

For the disposal of the product, obey the legislation of your country.

Keep out of reach of children.

This material has been manufactured for dental use only and must be handled and stored only in accordance with its instructions for use. The manufacturer is not responsible for any damage caused as a result of use not in accordance with the guidelines, improper storage or incorrect handling. Furthermore, the user is obliged to verify and verify, before handling and/or applying the product, and under his/her responsibility, if this material is compatible with the intended use. Furthermore, it should be noted that data descriptions do not constitute any kind of guarantee and, therefore, are not binding.

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