Diamond

BR | ES | EN | FR

DENTSCARE LTDA - Av. Edgar Nelson Meister, 474 - Distrito Industrial - 89219-501 - Joinville - SC - Brasil / Brazil. Autorização de funcionamento / Authorization of functioning: MS P5X44XY0XX28 - CNPJ / Tax ID: 05.106.945/0001-06 INDÚSTRIA BRASILEIRA / BRAZILIAN INDUSTRY Responsável Técnico / Technical Authority: Friedrich Georg Mittelstädt CRQ: 13100147 - SC - Registro na ANVISA / Registration at ANVISA nº 80172310002 - Marca / Brand: FGM®

NUNO FLORES - Al. Bonifacio Lazaro Lozano, 3 – Piso 0 – C 2780-125 Oeiras / Portugal - Tel: (351) 21 4439292 - ec.rep@fgm.ind.br

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Discos de feltro

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EN Instructions for use

Felt disks

Professional use only
Read all the information in this manual carefully before using the product. Keep the manual for later reference at least until the product is finished or until there is no more interaction between the product and its last patient.

Product description
Diamond felts are disks of natural felt used to support pastes for polishing surfaces. They have a quick fitting system which facilitates their connection to the chuck. Just a very light touch is needed to produce a high lustrous shine on porcelain and composites in combination with polishing pastes.

Presentation form
Package containing 24 felt disks (8 mm and/or 12mm) and 1 mandrel.

Basic composition
Disks: Special felt (wool) and silicone rubber.
Mandrel: Nickel-plated brass (dimensions as per ISO 6360/2 and ISO 1797/2).

Product indication
It is indicated for polishing dental restorations

in combination with Diamond polishing pastes.
Precautions and contraindications
Single-use product (disks). Discard after use.
Use at a low rotation and without exerting pressure on the disk during the polishing.
Check that the disk is well fitted in the chuck before activating the rotation. Rotations which are too high can cause the disk to be expelled from the chuck and excessive heating in the polishing area.

Instructions
1. Mount a disk onto the mandrel tightly and insert the shank completely into the chuck of handpiece.
2. Fit the unit (chuck and felt disk) in the counter angle and apply a small portion of polishing paste on the upper surface of the disk or directly on the surface to be polished.
3. Start the procedure using a low rotation, low pressure and intermittent movements on the surface to be treated. This avoids excessive heating of the surface.
NOTE: If you use pastes with different granulations, reserve different disks for each paste. Using the same disk will lead to failure in the final finishing as the result of the paste with the thickest granulation will prevail.
Conservation and storage
Keep the product in its original package always well sealed.

Warnings
Do not utilize the product after the expiry date. When disposing of the product, follow the legislation of your country. Keep out of children's reach.
This material was manufactured for dental use only and must be manipulated according to the manufacturer's instructions of use. The manufacturer is not responsible for any damage caused by misuse or improper manipulation of the product. The user is obligated to prove (before using the product and under his/her responsibility) that this product is compatible with the desired utilization, especially when such utilization is not indicated in the instructions of use. Data description does not constitute any type of warranty and therefore is not binding.

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