DENTSCARE LTDA - Av. Edgar Nelson Meister, 474 - Distrito Industrial - 89219-501 - Joinville - SC - Brasil / CNPJ / Tax ID: 05.106.945/0001-06 INDUSTRIA BRASILEIRA / BRAZILIAN INDUSTRY - Responsável Técnico / Technical Authority: Friedrich Georg Mittelstädt CRQ: 13100147 - SC, ANVISA nº 80172310071 - Marca / Brand: FGM® / Fabricado no Brasil / Made in Brazil

NUNO FLORES - Al. Bonifácio Lazaro Lozano, 3 - Piso 0 - C 2780-125 Oeiras / Portugal - Tel: (351) 21 4439292 - ec.rep@fgm.ind.br

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Instructions for use

Light-curing composite for occlusal stops and band cementation

For professional use only

Read carefully all the information in this manual before using the product. Keep it for later reference, at least until total consumption of the product and/or until there is no more interaction of the product with its last patient.

Product description

Ortho Bite is a light-curing self-adhesive composite with self-levelling characteristics and high level of surface hardness. Its composition includes dimethacrylate monomers, HEMA Phosphate, traditional Camphorquinone/DABE system as photoinitiator and silica as load particle. The combination of silica load fillers with the dimethacrylate monomer composition results in a high viscosity that allows it to be handled and applied without running off the application area. The presence of HEMA Phosphate allows good adhesion to the enamel surface, which is a known feature of composite cements indicated for bracket bonding. The thixotropy allows the product to have self-levelling capability, known as a shear thinning property that makes the composite settle correctly on the occlusal surface leaving no sharp edges on its surface. The product is also colored in order to facilitate its identification during application and removal.

Commercial Availability

Check the availability in your country.
- Packaging containing 1 syringe of 4g of the product.

Basic Composition:

Active ingredients: dimethacrylate monomers, methacrylate monomer phosphate, stabilizers, camphorquinone and coinitiators.
Inactive ingredients: silanized silicon dioxide inorganic load filler and dye.

Product indication

The product is indicated for cementing orthodontic bands and is also suitable for temporary dental disocclusion during orthodontic treatment.

Application description

When applied on the occlusal area, the composite works as stop allowing the occlusal contact to occur only on this tooth and its antagonist during centric relation (CR*) or maximum intercuspation (MIH**), inhibiting intercuspation of the other teeth. The product is of temporary nature, aiming to aid on the treatment's mechanics and should be removed at a certain stage of the treatment, according to the case planning.

Due to the high bond strength to dental enamel, the product is also suitable for cementing orthodontic bands.

*CR is a maxillo-mandibular relationship in which the condyles articulate with the thinnest avascular portion of their respective discs with the complex in the anterior-superior position against the slopes of the articular eminences. In practice, it is generally the first dental contact when the patient closes his/her mouth.

**MIH is the point where there is the greatest number of dental contact when the patient closes his/her mouth.

Precautions

Professional use only;

Avoid contact of the product to eyes and skin. Wear gloves to manipulate the product;
In case of contact with the eyes, wash immediately with water in abundance. If there is persistent irritation, search for medical support;

If the product contacts skin, rinse the area with water and soap. If necessary, search for medical support;
Wear protective goggles during the light activation of the product;
Do not expose the material to high temperatures or intense light.

Avoid contact of phosphoric acid (etchant) to mucosa, skin and eyes. It can cause burns. Wear gloves for its manipulation;

Do not store the product near eugenol-based materials because those can inhibit the adequate cure of adhesive/cement.

Side effects

As this is a product that contains well-known monomers and since it is indicated to be used only on enamel (noninvasive procedure), no side effects are expected (considering proper use of the product).

Instructions for use

Note: The instructions for use are listed below. However, it is up to the professional to judge the best way to apply the product, taking into consideration each clinical case. The quantity and shape of composite to be applied, the area of application as well as how long should the composite remain on the tooth can vary significantly.

For occlusal stops:

1. Select the tooth (teeth) to receive the occlusal stop according to the case planning.
2. Proceed tooth prophylaxis of the occlusal area. Once clean, dry the area.
3. Acid etch the surface with phosphoric acid at 37% for 30 seconds. After this period, rinse and dry the area, maintaining it isolated and free of contaminants by relative or total dam isolation.
4. Apply Ortho Bite on the occlusal surface of the tooth in 2mm increments until achieving the correct level of disocclusion.

Note: The height of the increment is generally the smallest as possible to produce disocclusion of about 1mm to other teeth. Exceeding height can produce excess of force leading to occlusal dysfunctions and pain.

5. Light cure each increment for 20 seconds with a light-curing unit of minimum power of 600mW/cm².

Note: When concluding, check with articulating film.

7. This composite is of temporary nature. Considering the planning perspective, as soon as its intended goal is fulfilled, the composite must be removed by using multi-fluted burs.

Note 1: if there is need for additional composite increments, this must be performed as following:

1. Dry the area and isolate it from fluids.
2. Apply dental adhesive on the area to receive the increments. Apply the first layer actively during 10 seconds and then a new layer must be applied for another 10 seconds as well. Then, apply air jet gently for 10 seconds and light cure it for 10 seconds.
3. Apply Ortho Bite according to the need and light cure it for 20 seconds each increment. The occlusion contacts must be checked again.

Note 2: the composite is self-adhesive when applied to etched enamel, and the use of dental adhesive is unnecessary. Not using the dental adhesive provides the composite sufficient bond strength to support most cases. However, in cases where the occlusal force is too high (e.g. parafunctional habits, overbite, braquiacial patients, etc.) or the occlusal surface shows deep and irregular pits and fissures, the dental adhesive must be used to increase bond strength, once it penetrates through the pores created by the previous surface etching. Refer to the adhesive's instruction manual on how to apply it.

For orthodontic bands cementation:

1. Perform local tooth prophylaxis with a Robinson brush, water and pumice.

2. Acid etch buccal and lingual/palatral surfaces of the tooth with phosphoric acid at 37% during 30 seconds. Then, rinse and dry it completely.

3. Isolate the area correctly for it to be free of fluids.

4. Roughen the internal surface of the orthodontic band with the aid of an abrasive bur for better composite retention.

5. Apply a small amount of Ortho Bite all over the band's internal perimeter and fit it to the tooth, making sure it is fitting correctly (if necessary use a bite stick).

6. Remove exceeding cement that overflows with a clinical probe or spatula.

7. Light cure it for 40 seconds on the occlusal surface and 40 seconds for the cervical surface (considering light-curing unit of minimum power of 400mW/cm²).

Note: ask the patient not to eat or drink for a period of 30 minutes after cementing the orthodontic band.

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