DENTSCARE LTDA - Av. Edgar Nelson Meister, 474 - Distrito Industrial - 89219-501 - Joinville - SC - Brasil/ Brazil. CNPJ/ Tax ID: 05.106.945/0001-06
INDÚSTRIA BRASILEIRA / BRAZILIAN INDUSTRY - Responsável Técnico/ Technical Authority: Friedrich Georg Mittelstädt CRQ: 13100147 - SC - ANVISA nº 80172310043 - Marca / Brand: FGM® / Fabricado no Brasil / Made in Brazil

NUNO FLORES AL. Bonifácio Lazaro Lozano, 3 – Piso 0 – C 2780-125 Oeiras / Portugal - Tel: (351) 21 4439292 - ec.rep@fgm.ind.br

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EN Instructions for use

Cement/adhesive for orthodontic brackets fixation

For professional use only

Read all the information in this document carefully before using the product. Keep this document for future reference.

Product description

Orthocem is a light-curing orthodontic single syringe cement that promotes the union between ceramics or metal brackets to the enamel surface. It has a fluorescent substance that allows tracing the product when applying ultraviolet light, facilitating product´s removal from teeth without damaging enamel, after removing brackets.

Commercial availability

Check the availability in your country.
- Package containing 1 syringe with 4g of the product, 1 syringe with 2,5 mL of phosphoric acid (Condac 37) and 1 tip.
- Package containing 1 syringe with 4g of the product.

Basic Composition:
Contains methacrylate monomers (BisGMA, TEGDMA) and methacrylate phosphatized monomers, stabilizers, canforoquinone, co-initiators, silicon dioxide loading and pigment.

Product indication

Cementation of metal, ceramic or polycarbonate brackets or tubes to enamel surface.

Precautions

Professional use only.
- Avoid contact of the product to eyes and skin. Wear gloves and goggles to manipulate the product.
- In case of contact with the eyes, wash immediately with water in abundance. If there is persistent irritation, search for medical support.
- If the product contacts skin, rinse the area with water and soap. If necessary, search for medical support.

- Wear protective goggles during the light activation of the product.
- Do not expose the material to high temperatures or intense light.
- Avoid contact of phosphoric acid (etchant) to mucosa,skin and eyes.It can cause burns. Wear gloves for its manipulation.
- Do not store the product near eugenol-based materials because those can inhibit the adequate cure of adhesive/cement.

Side effects

The product contains metacrylate monomers and since it is indicated to be used only on enamel (non-invasive procedure), no side effects are expected (considering proper use of the product).

Instructions for use

Before initiating the treatment, read carefully the Precautions, Contraindications and possible Side effects.

1. Isolation of operatory field: the operatory field must be kept free of saliva contamination, by means of an efficient isolation. Additionally, it is recommended to use a lip retractor for better access to the teeth.
2. Teeth preparation: proceed to teeth prophylaxis with oil free pastes. Rinse with water in abundance and dry the teeth with air free of oil and humidity.
3. Acid etch: apply 37% phosphoric acid following manufacturer's recommendations.
4. Rinse with water in abundance and air dry.
5. Apply the adhesive system in accordance with the manufacturer's recommendations.
6. Bracket bonding: apply little amount of Orthocem straight from the syringe onto the base of the bracket and put it on position on the teeth.
7. Remove the excess of cement without displacing it from its correct position.
8. For bonding brackets made of metal, ceramic or polycarbonate, light cure Orthocem for 40 seconds (*) with the tip of the light-curing unit as near as possible to the bracket´s margins (10 seconds for each margin).
9. The orthodontic arch wire can be placed immediately after bracket bonding.
10. After concluding the orthodontic treatment, brackets must be removed with the aid of specific instruments. The remaining cement adhered to dental surface can be removed with multi fluted special burs and fine sandpaper disks, using UV light (**) to identify where exactly the remaining cement is, in order to avoid damaging dental enamel. Although the extra fluorescent capability of the cement allows it to be easily identified under UV light, the dentist must remove excess carefully.
11. Then, the teeth should be polished with felt disks and polishing paste of extra-fine grain.

*Note: The activation time mentioned above considers that the light-curing unit has minimum power of 400 mW/cm². Equipments with less power should have extended light-curing periods. It must be used blue light with a wavelength of 400 to 500nm.

**Note: Considering UV light source with intensity of 250 mW/cm² and wavelength of 390 to 410nm.

Conservation and storage
Do not expose the product to high temperaturesor to light.
Do not store the material next to other containing eugenol because it can inhibit the adequate polymerization process of the adhesive.
Store the product at a temperature between 5 –30° C /41 – 86 °F.

Expiration date
Printed on the product packaging.

Warnings

Do not use the product after its expiry date. For disposal follow your country´s legislation. Do not reuse the empty package. Keep out of children´s reach. Avoid the contact with skin and eyes. This product contains acrylate monomers, which are known to produce allergic skin reactions in certain sensitive individuals. May cause eye and skin irritation, thus the use of protective gloves and goggles while manipulating the product is mandatory.

This material was manufactured for dental use only and must be manipulated according to the manufacturer's instructions of use. The manufacturer is not responsible for any damage caused by misuse or improper manipulation of the product. The user is obligated to prove (before using the product and under his/her responsibility) that this product is compatible with the desired utilization, especially when such utilization is not indicated in the instructions of use. Data description does not constitute any type of warranty and therefore is not binding.

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