DENTSCARE LTDA - Av. Edgar Nelson Meister, 474 - Distrito Industrial - 89219-501 - Joinville - SC - Brasil / Brazil - CNPJ / Tax ID: 05.106.945/0001-06 INDUSTRIA BRASILEIRA * BRAZILIAN INDUSTRY - Responsável Técnico / Technical Authority: Friedrich Georg Mittelstädt CRQ: 13100147 - SC / ANVISA nº 80172310088 - Marca / Brand: FGM® / Fabricado no Brasil / Made in Brazil

NUNO FLORES AL. Bonifacio Lazaro Lozano, 3 – Piso 0 – C 2780-125 Deiras / Portugal - Tel: (351) 21 4439292 - ec.rep@fgm.ind.br

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Universal chroma light-curing composite

For professional use only

Read all the information in this document carefully before using the product. Keep this document for later consultation at least until the product lasts or until there is no more interaction between the product and its last patient.

Product Description

Universal chroma light-curing composite for all dental shades for use in anterior and posterior restorations, recommended for all classes of cavities. The composite is radiopaque, with total inorganic load of 72% to 80% in weight (52% to 60% in volume). Those features contribute for both mechanical and aesthetic characteristics, especially those evidenced by the easiness to obtain long-lasting polishing. It does not contain Bis-GMA nor Bis-EMA in its formula, following the present trend of Bisphenol A (BPA) free products. The composite has APS as its photoinitiation system. APS stands for Advanced Polymerization System, and consists of a combination of different photoinitiators that interact among themselves amplifying the curing capacity of the light emitted by the light-curing device. Added to different materials, the system offers different advantages. APS offers great polymerization strength allowing for a higher conversion rate, obtaining superior stability under room light and allowing more time for restorations.

Basic Composition

Active ingredients: mixture of methacrylate monomers, photoinitiator composition (APS), co-initiators, stabilizers and silane. Inactive ingredients: boron-aluminum-silicate glass.

Commercial availability

Refill: 1 4g-syringe. Single-dose capsules: package containing 20 capsules of 0.2g each.

Product Indication

The composite is indicated for use in permanent and deciduous teeth:

- Direct restorations in anterior and posterior teeth (Classes I, II, III, IV, V and VI).

Warning

- This product is recommended to be used only according to its instruction manual;
- Avoid the contact of the non-photopolymerized composite with the skin, mucosa or eyes because it can be irritating due to the presence of methacrylate. The use of all the protection equipment is recommended during the manipulation of the product;
- Avoid the use of materials containing eugenol, such as temporary filling or restorative materials because they can affect the correct curing of the composite;
- When restoring large cavities, filling materials (like calcium hydroxide) must be used in the pulpal wall;
- The composite is sensitive to light, therefore, make sure you close the syringe well after using. The composite increment must be used right after it is out of the tube;
- Use protection goggles when using the light-curing device;
- The product must not be swallowed or aspirated. In case it is, search for medical help;
- The product is not recommended if it is not possible to have a completely dry operation field or if the restorative technique cannot be followed.

Contraindications

- Do not use the product if the patient is allergic to any of the substances in the composition. In case of allergic reactions, discontinue the use of the product, and, if necessary, search for medical help.

Side effects

This product contains methacrylate monomers that may cause allergic reactions in patients that are sensitive to those substances.

Instructions for use

Read carefully the Warnings, Precautions and Contraindications and possible Side Effects in this document prior to the use of the product. Product application

1. Isolate the operatory field. The isolation must guarantee a field free of moisture with good visualization. If necessary, provide absolute isolation.
2. Prepare the cavity according to the adhesive technique.
3. Perform the acid etching of the tissues (enamel and dentine). Afterwards, rinse abundantly removing the acid from the surface and dry the enamel gently, leaving the dentine moist.
4. Apply the adhesive according to the manufacturer's instructions.
5. Insertion of the composite and photopolymerization*. In case there is exaggerated darkening due to the formation of tertiary dentin, staining from amalgam or other staining with no known origin, it is necessary to apply the composite in the shade dentin (ex.: Vittra APS, shades DA1, DA2, etc.) in layer of 1-2 mm to mask the darkening (as an option, a composite of the

Opacificador type can be used). For the insertion of the restorative composite, the incremental technique is recommended because it allows for better performance of the material. Increments of up to 1.5 mm must be inserted, adjusted and photopolymerized for the time recommended in the cavity.

*The composite will only mimic the color of the tooth if there are walls in the surroundings, and after polymerization. During the insertion and sculpting, the composite shows more opacity and will only resemble the color of the tooth after photopolymerization.

6. Fill in the cavity in increments until concluding the restoration.

7. An oxygen blocking agent may be used in the most superficial layer of the restoration (last increment) in order to isolate the composite from the air, allowing for the complete curing of the composite and improving its shine.

8. For finishing, use Diamond Pro (FGM) sand disks or fine granulation drills and for polishing use Diamond Flex felt disks (FGM) with Diamond Excel paste (FGM). Note: the checking of the occlusion is important. Restorations in supra-occlusion cause excessive stress on the tooth and restoration, which may lead to pain. Do not leave restorations in infra-occlusion.

Single-dose capsule:

1. Insert the capsule inside the syringe system in the correct manner so that it is ready for use.
2. Remove the cap of the capsule and apply the composite in the cavity with manual pressure on the application syringe/pistol.
3. After the restoration is done (in increments), discard the capsule.

Application Guidelines by Cavity Class

- Class I, II, V and VI cavities: in case of darkening, cover the area with dentin resins or opacifiers (according to the professional's choice) and proceed with the incremental technique restoration with Vittra APS Unique.
- Class III and IV cavities (without a back wall to support the color): make a layer that will serve as a background for the restoration and finish the rest of the volume with Vittra APS Unique incrementally.

Photopolymerization times

Incremental technique with 1.5mm layers with photopolymerization for 20 seconds* per increment.

*Considering the light-curing equipment with capacity over 450mW/cm2 and wavelength of 400 to 500nm.

Conservation and storage

The product must be kept in its original package, closed.

Protect the product from direct sunlight; Keep the product at temperatures between 5°C and 30°C/41°F and 86°F.

Note: if the product is stored in a refrigerator, wait, preferably, about 15 minutes before using it. The interval between 20 and 25°C is the best temperature for applying the product.

Warnings

- This material was manufactured for dental use only and must be handled according to the instructions for use;
- Do not use the product after its expiration date;
- Follow the legislation of your country for discarding

the product;
- Keep out of the reach of children.
Expiration date
Printed on the product packaging.

The manufacturer is not responsible for damage caused by misuse, improper storage or incorrect manipulation. The user is obliged to prove (before handling and/or applying the product, and under his/her responsibility) that this product is compatible with the desired usage, especially when the use is not recommended in the instructions for use. Data description does not constitute any type of guarantee and, therefore, have no binding.

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